

BOTTLE FEED SYRINGES

LD.3EM-BT

(Adjustable dosage from .1ml to .3ml)

Product Code: LD.3EM-BT



2EM-BT

(Adjustable dosage from .25ml to 2ml)

Product Code: 2EM-BT



5EM-BT

(Adjustable dosage from .5ml to 5ml)

Product Code: 5EM-BT



- Automatic self-filling with bottle attachment
- Heavy-Duty Bottle Cage for bottle protection (50ml & 100ml bottles only)
- Accurately delivers selected dose
- Easy V-grip for high volume use
- Metal Luer Lock tip
- Removable barrel for cleaning
- Durable plastic design
- Amber-colored barrel reduces UV sunlight exposure



Allflex No.	Description
LD.3EM-BT	.3ml Low Dose Automatic Reusable Bottle Top Syringe
2EM-BT	2ml Automatic Reusable Bottle Top Syringe
5EM-BT	5ml Automatic Reusable Bottle Top Syringe

USING A BOTTLE FEED SYRINGE

To attach a bottle to the top-load syringes, models 2UM and 5FM:




Unscrew the bottle nut two turns only



Carefully remove the safety cap




Slide the bottle into the bottle holder (Make sure bottle is firmly seated)

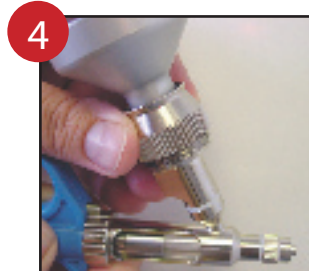

Tighten swivel cap and secure bottle



Loosen dose adjustment lock nut

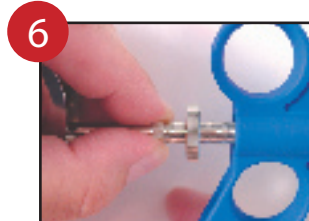

Rotate dosing adjustment to set the required dosage. Retighten lock nut.